



Smart Fixation Bands for Peripheral IV Catheter with IV Set & IV Catheter with Heparin cap

- Safe securement for IV catheter hub
- Safe holders prevent IV Cath kink ,disconnection, movement
- Save cost by prolonging IV Cath dwell time
- Reduces IV Cath related complications such as Phlebitis, occlusion
- Easy application(Press-Peel-Place)

Needle Lock II, III, H

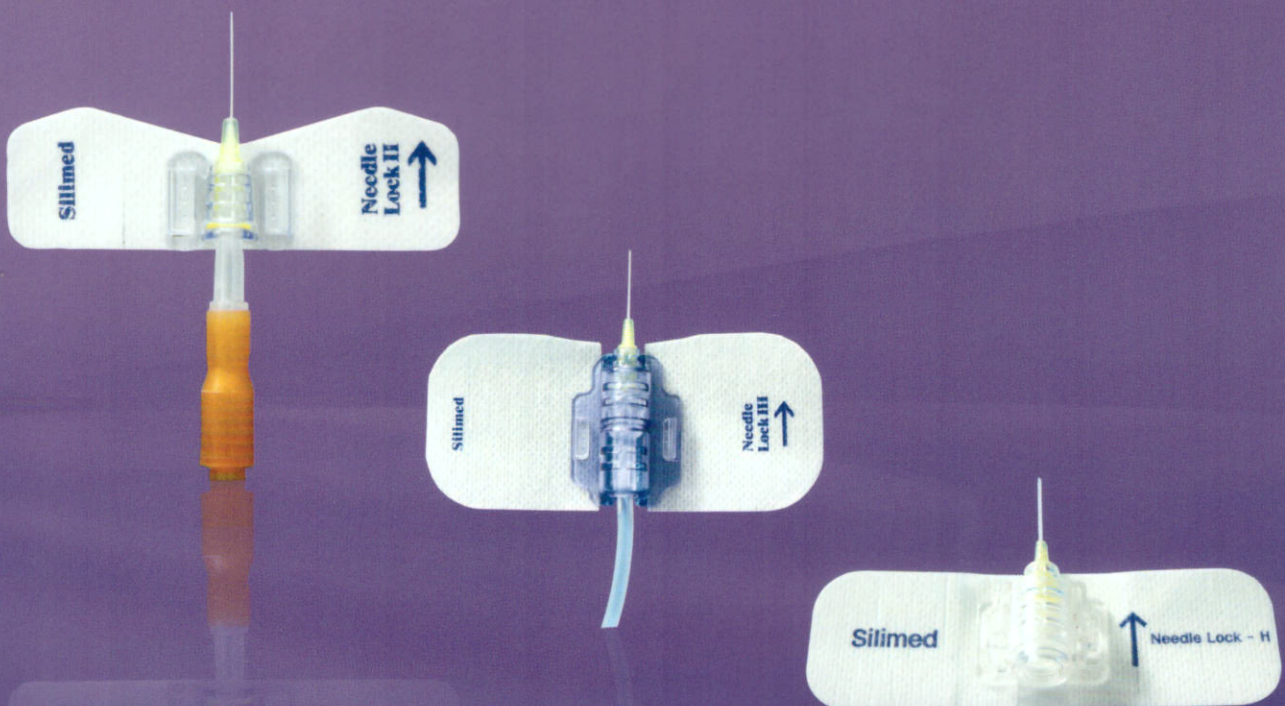

Needle Lock Smart IV Catheter Fixation Bands

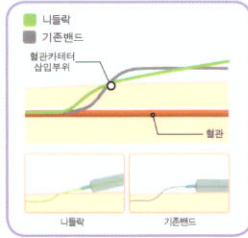

Safe Securement for IV Catheter hub

- ☞ 카테터 허브를 고정하여 혈관내 카테터의 완벽한 각도를 유지
- ☞ 안전고정대로 카테터 허브를 고정해 카테터 꼬임, 빠짐, 미세움직임 감소
- ☞ 인정비급여코드 BJ002CXV




정맥염



혈관폐색

Features

- ☞ "CDC (2011) Guide lines에 의해 말초정맥관 카테터를 최대 72~96시간까지 유지가능" 
- ☞ IV 카테터 재삽입률 감소
- ☞ IV 카테터 삽입 관련 합병증을 줄임(정맥염, 혈관폐색)
- ☞ 사용이 쉬움(누른다-떼어낸다-부착한다)

Needle Lock II, III, H

Instructions For Use



01 IV 카테터 삽입부위에 직사각형 밴드를 부착한다.



02 니들락 고정대를 IV 카테터 허브 위에서 수직으로 눌러 고정시킨다.



03 니들락 고정대를 잡은채로 양면면의 점착지를 떼어낸 후 피부에 부착한다.



04 수액라인을 고정대 양면 고리에 걸쳐 수액라인을 정리한다.



05 여분의 부직포 밴드로 수액라인을 말끔히 정리한다.