


REGIONAL ANAESTHESIA

Ultrasound and nerve stimulation


Peripheral nerve blocks

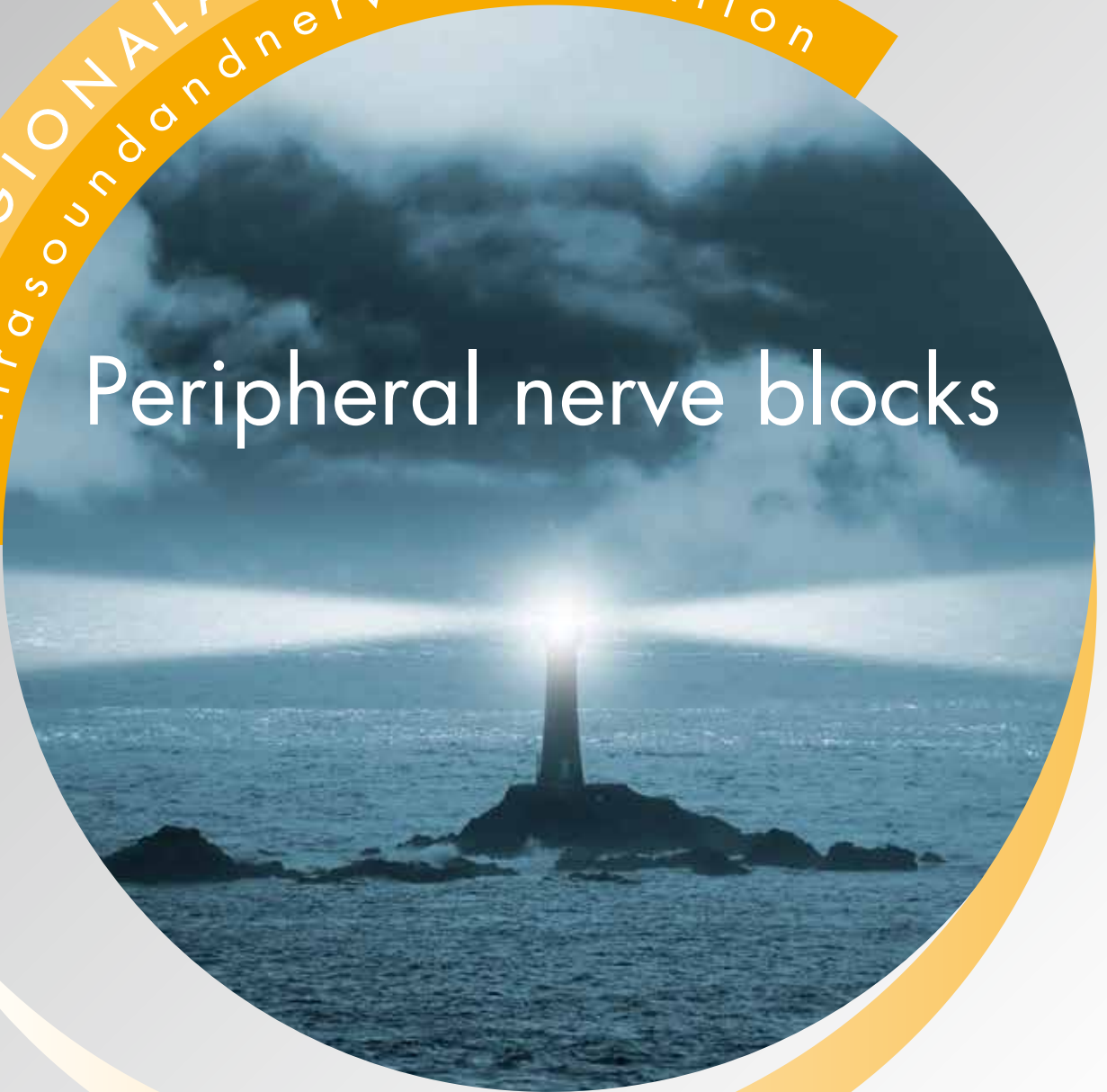

Value Life

Echogenic and stimulating catheter Continuous injection

silverstim combines modern techniques to offer the most precise placement of catheter. Thanks to a new coating method, a thin silver layer is spread on the whole length of the catheter. This ensures optimum conductivity and echogenicity, without reducing the internal diameter.

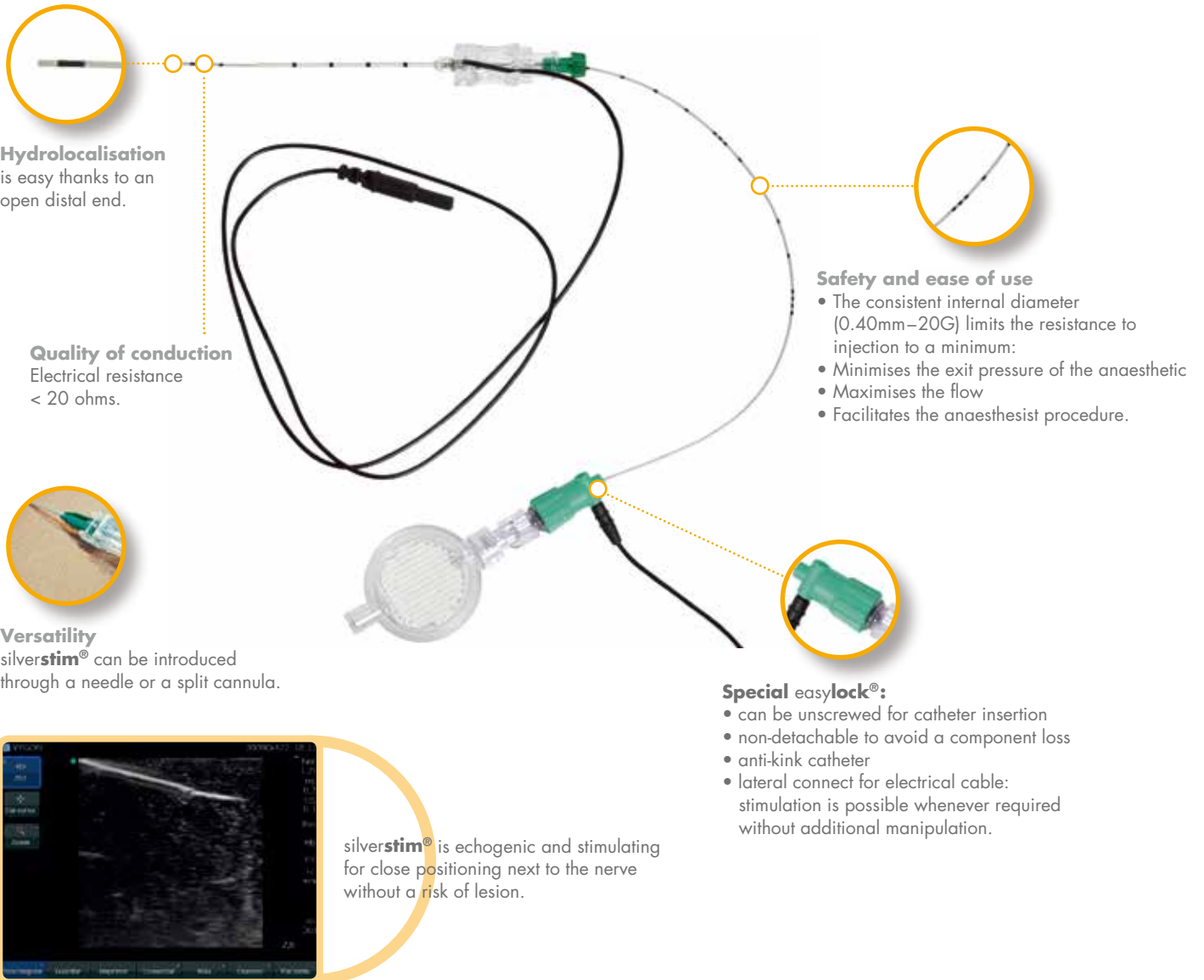

Product Code	30° Needle 길이	Cath 길이	Needle Gauge	주사용 Indication
5198.503	50mm	300mm	18G	Interscalene & Femoral
5198.853	85mm	500mm	18G	Sciatic
5198.123	120mm	500mm	18G	Sciatic

Single use pump for Continuous Nerve Block



CNB CARE

통증의 변화 양상에 따라 약물의 적정 주입용량을 선택, 조절할 수 있도록 7단계로 설정하였으며, 필요 시 기본 약물주입을 차단(Off)할 수 있고, 환자의 요구에 따른 추가 주입(Bolus)을 할 수 있습니다.



Variable Module & Key

기본주입 유속을 설정할 경우

유속조절키를 모듈부에 삽입하고, 7단계의 유속과 off모드 중 정확한 위치에 오도록 용량을 선택, 설정한 후 유속조절키는 제거하여 별도로 보관합니다. 공기를 제거한(Priming)후 반드시 원하는 유속에 화살표의 방향을 맞추어야 합니다.

Technical Specification

■ 제조조건(Production Condition)

- 약물저장부(Balloon reservoir)와 유속조절부는 수평선상에 위치, 23°C, 0.9%N/S의 기준에서 제조, 생산합니다.

■ 유속 정확도(Flow-rate Accuracy)

- Reservoir Volume에 따른 정확도 : 120ml / 180ml : ±10% / ±15%
- 사용 약물의 온도(Temperature), 점도(Viscosity)에 따라 기본 유속은 달라질 수 있습니다.
- 높이(Height)에 따른 중력의 영향은 유속에 영향을 줄 수 있으며, 약물 저장부와 유속조절부의 수평선상 위치에서 유속은 가장 안정적입니다.

Note

펌프를 고정할 경우에는 높낮이의 차이를 최소화 하는 것이 유속의 오차를 줄일 수 있습니다.



Product Specifications & Code Information

	Volume	Basal flow rate	Bolus Volume	Lockout time
Product Specifications	120ml (BN)	0.5ml 단위 : off, 0.5 ~ 3.5ml/hr	0.5ml, 1.0ml, 2.0ml	15min
	180ml (BR)	1.0ml 단위 : off, 1.0 ~ 7.0ml/hr		
Code Information	BN03515R	120ml	0.5ml	15min
	BR1715E	180ml	1.0ml	

[수입원] P&F CARES (주) 피엔에프케어즈
경기도 성남시 분당구 판교로 255번길 35
Tel. 031-8023-9970 Fax. 031-8023-9971

For further information, please contact: questions@vygon.com

The specifications shown in this leaflet are for information only
and are not, under any circumstances, of a contractual nature.

Vygon – 5, rue Adeline • 95440 ECOUEN • FRANCE
Reception: +33 (0)1.39.92.63.63 – Service clients France: +33 (0)1.39.92.63.81
Export customer service: +33 (0)1.39.92.64.15
Fax: +33 (0)1.39.92.64.44 • www.vygon.com

